



Summary of sample collection, storage and transport instructions for GenID tests

| Sample tube or swab name | Manufacturer | Storage | Stability | Transport | Valid for |
|-----------------------------------|---|---------|-----------|-------------------------|-------------------|
| CyMol | Copan | 2-30°C | 3 weeks | ground only, flammable | STD+HPV |
| Easyfix | Labonord | 2-30°C | 3 weeks | ground only, flammable | STD+HPV |
| LiquiPrep | LGM International Inc. | 2-30°C | 1 week | ground only, flammable | STD+HPV |
| SurePath | Becton, Dickinson and Company | 2-30°C | 3 weeks | ground only, flammable | STD+HPV |
| PreservCyt | Hologic Corp. | 2-30°C | 3 weeks | ground only, flammable | STD+HPV |
| cobas® PCR Cell Collection Media | Roche Molecular Systems Inc. | 15-30°C | 6 weeks | ground only, flammable | STD+HPV |
| cobas® PCR Female Swab Sample Kit | Roche Molecular Systems Inc. | 2-30°C | 3 weeks | air or ground transport | STD+HPV |
| cobas® PCR Urine Sample Kit | Roche Molecular Systems Inc. | 2-30°C | 3 weeks | air or ground transport | STD |
| buccal swab (brush type only) | any CE marked | 2-30°C | 3 weeks | air or ground transport | all genetic tests |
| forensic dry swab | Copan Flocked Swab 4N6, snap off, 2 ml eppendorf tube with evaporation-cap, peelpack, sterile; Code: M10181 | 2-30°C | 3 weeks | air or ground transport | all genetic tests |
| cervical swab or brush | any CE marked for cervical sampling | NA | NA | NA | STD+HPV |
| urethral swab or brush | any CE marked for urethral sampling | NA | NA | NA | STD+HPV |
| urine sample collection tube | any sterile CE marked | NA | NA | NA | STD |
| Qvintip® | Aprovix | 2-30°C | 1 week | air or ground transport | HPV |